




Reduce risk. Improve outcomes.

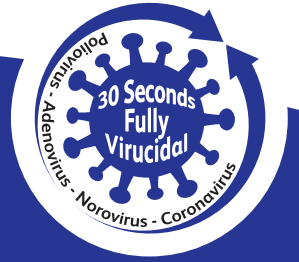

Oxivir® Excel Wipe

Patented accelerated hydrogen peroxide based cleaner and disinfectant.

Oxivir® disinfectant cleaners, powered by AHP, rewrite the rules in disinfection by delivering broad efficacy, short contact times and superior cleaning with formulations that also have excellent user and surface compatibility profiles and a sustainable active ingredient.

- Ready to use, cleaner and disinfectant wipes in one for super cleaning and faster disinfection
- Active ingredient in AHP based Oxivir wipes breaks down into water and oxygen leaving no harmful chemicals or residues on the surface
- Disposable wipe for ultra hygiene
- Convenient, no preparation or laundry required
- Kills MDRO's such as MRSA, CRE, VRE, CRPsA
- Effective against Bacteria, Viruses and Yeast in as little as 30 seconds
- Full virucidal claim including enveloped viruses such as Coronavirus and non-enveloped viruses such as Norovirus & Poliovirus in 30 Seconds in Dirty Conditions
- Extensively tested against the new European norms
- Carries yeasticidal claim against Candida albicans
- Powerful surfactant package delivers excellent cleaning performance including descaling and lime limitation
- Excellent user safety profile reduces worker risk and absence
- Does not contain respiratory irritants
- Does not contain Salicylic acid or quats
- AHP formulation is gentle on surfaces extending equipment life
- CEIIa certification allows use on for non-invasive medical devices
- Also available in a RTU or a liquid concentrate
- Suitable for use in high risk areas including ICU, OR, NICU and patient rooms
- No CLP Hazard classifications or statements - hence no need for PPE or gloves



Bactericidal



Virucidal



Yeasticidal

ACCELERATED
HYDROGEN PEROXIDE



Surface Compatibility



Oxivir Excel Wipe can be used on any hard surface usually encountered in a healthcare environment including stainless steel, glass, acrylics and PVC. It is not suitable for use on marble, copper, brass or other surfaces sensitive to oxidative agents.




Oxivir[®] Excel Wipe


Patented accelerated hydrogen peroxide based cleaner and disinfectant.

Oxivir Excel Wipes				
	EN test	Organisms		RTU Wipes
Bactericidal	EN 1276: 2010 FIDI	<i>E. coli</i> , <i>S. aureus</i> , <i>P. aeruginosa</i> , <i>E. hirae</i>	Dirty	1 min
	EN 13727:2015 MED		Dirty	30 secs
	EN 13697: 2015 FIDI		Dirty	5 mins
	EN 16615: 2015 MED		Dirty	1 min
Virucidal	EN 14476: 2013 MED	<i>Adenovirus / Poliovirus</i>	Dirty	30 secs
		<i>Murine Norovirus</i>	Dirty	30 secs
	EN 14476: 2013 MED	Fully Virucidal	Dirty	30 secs
Yeasticidal	EN 1650, EN 13624, EN 13697	<i>C. albicans</i>	Dirty	5 mins
	EN 16615: 2015 MED		Dirty	1 min
Fungicidal	EN 1650, EN 13624, EN 13697, EN 16615	<i>Aspergillus Niger</i>	Dirty	15 mins


How to Use

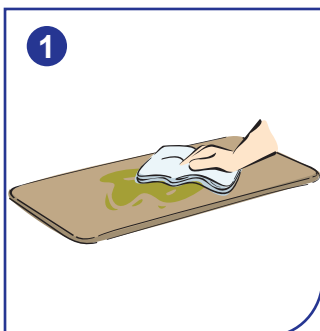


1 Remove excessive debris from the surface.

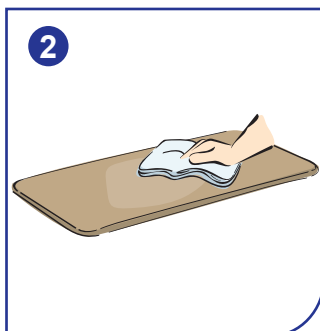

2 Using the Oxivir Excel wipe clean and disinfect all the surfaces.

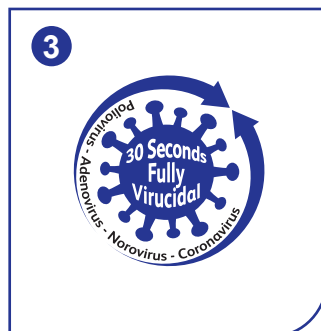

3 Allow 30 seconds to meet full virucidal claim against Coronavirus, norovirus & poliovirus in dirty conditions.

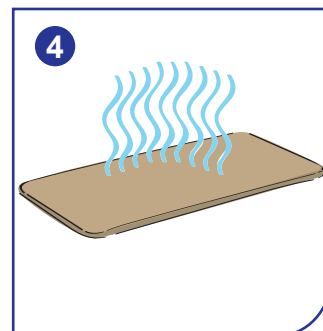

4 Allow to air dry.

Composition

100 gram product contains:

Active Ingredients: peroxide 0.36%.

Available Pack Formats

- 12 x 100pc wipe flow wrap pack
- 12 x 30pc wipe flow wrap pack
- Wipe size 267x200mm (1.64 sqm coverage)
- 6x160pc Canister Wipe pack
- Wipe size 170x150mm (2.24 sqm coverage)

All packs are ready to use product as indicated above.

Substrate: 100% polypropylene

Authorisation and References



Oxivir Excel is certified as MDD - CE Class IIa medical devices (according the Medical Device Directive).

Storage

Store in accordance with local and national regulations. Keep only in original container. Store in a closed container. Keep from freezing.

Oxivir Excel Wipes provide ultra hygienic disposable disinfection with a safety profile that makes it easy to use to drive cleaning compliance.